

VETERINARY LARYNGOSCOPES



CONVENTIONAL LARYNGOSCOPE

FIBER OPTIC LARYNGOSCOPE

FEATURES



LED illumination-strong, focused and reliable LED light source.

Cool LED light source at the tip of the blade designed to lower the risk of damage to delicate oral tissues.

Brushed stainless steel with matte finish helps reduce glare during use.

Durable stainless-steel construction from tip to base designed for reliable performance.

The simple "click" locking and unlocking mechanism allows it to be attached quickly and reliably.

MATERIAL



Our Fiber optic laryngoscope blades are manufactured from a high quality AISI 304 stainless steel Steel Melt Type which is highly resistant to corrosion. The bulb is in the handle, blade can be sterilized at 134 °C; All blades and handles are removable and interchangeable with other blades of the same size.

APPLICATION

Bulb anesthesia laryngoscope is used to expose the airway during endotracheal intubation under general anesthesia, used in conjunction with a ventilator or anesthesia machine. Different models are available for different sizes of animals.

OPERATING INSTRUCTIONS


1. Engage the blade by aligning the slot of the blade on to the hook pin of the handle and apply a sufficient force ($\sim 1\text{lb}$) as shown in Fig.1.
2. Apply force upward to bring the blade into operating position as shown in Fig.2.
3. To bring the blade into standby position apply force downward as shown in Fig.2.

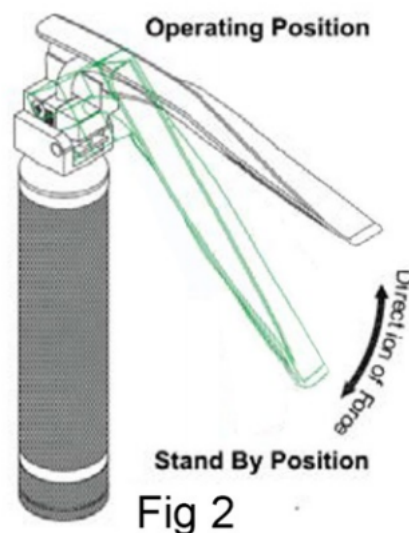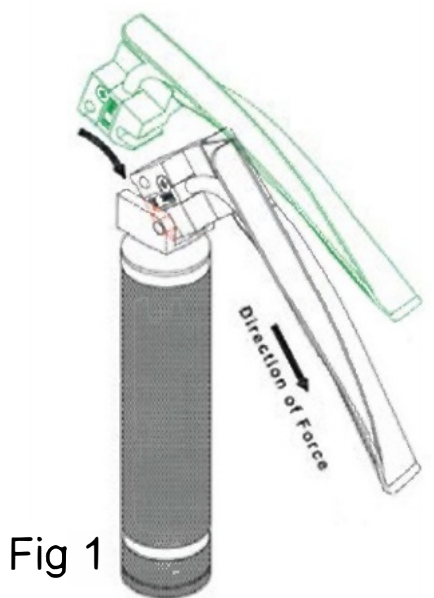

CONVENTIONAL AND FIBER OPTIC LARYNGOSCOPE

Model #	Description	Set Size
CL-02	Conventional Laryngoscope with bulb Reusable blades and handle 2 C-Type batteries (Batteries not supplied)	Set of 0, 1, 2
CL-04	Conventional Laryngoscope with bulb Reusable blades and handle 2 C-Type batteries (Batteries not supplied)	Set of 0, 1, 2,3,4
FL-02	Fiber Optic Laryngoscope Reusable blades and handle2 C-Type batteries (Batteries not supplied)	Set of 0, 1, 2
FL-04	Fiber Optic Laryngoscope Reusable blades and handle2 C-Type batteries (Batteries not supplied)	Set of 0, 1, 2,3,4

SIZE OF LARYNGOSCOPE MILLER

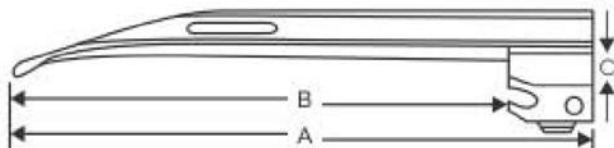

BLADE				
Size	A	B	C	Working Length
0	75mm	53mm	26mm	55mm
1	111mm	80mm	26mm	80mm
2	151mm	134mm	28mm	130mm
3	188mm	167mm	28mm	170mm
4	203mm	180mm	28mm	180mm

FIBER OPTIC LARYNGOSCOPE



Fig.4



Fig.3



Fig.2



Fig.0



Fig.1

CONVENTIONAL LARYNGOSCOPE



Fig.4



Fig.3



Fig.2



Fig.0



Fig.1